# RUCK® Fußwannenwagen ROOM POOL

## RUCK® Foot Bath Trolley ROOM POOL

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fon +49 (0)7082. 944 22 55  
fax +49 (0)7082. 944 22 59

E-Mail [service@hellmut-ruck.de](mailto:service@hellmut-ruck.de)

### Risk of damage due to improper care of surfaces

Disinfect the contact surfaces after each treatment with RUCK® surface disinfection wipes, alcohol-free (item no. 2967002, item no. 2967102 or item no. 2967202), medical product.

Please observe the exposure times of the disinfectants according to the product labelling. One minute for bacteria, five minutes for fungi. Further information on the disinfectant.

Danger of damage due to improper care of surfaces. Do not use any concentrated disinfectants, solvents, cleaning agents or liquids that contain alcohol (> 50 % alcohol content).

Take great care to ensure that the cleaning utensils used are clean. If mineral contaminants such as grains of sand or dust are embedded in the cleaning agents, this can destroy the surface. Do not use any scouring agents, such as abrasive cleaners, cleaning sponges or similar items that may contain abrasive particles. Do not use any scouring agents, such as abrasive cleaners, cleaning sponges or similar items that may contain abrasive particles. Residues of leaked fluids must be removed from the surface immediately with a soft cloth.

For gentle cleaning of the surfaces, use a soft cotton cloth or the RUCK® microfibre cloth which is gentle on surfaces (item no. 29616).

Caution: Cleaning cloths with plastic fibres can have an abrasive effect and consequently they can damage the surface.

Over time, traces of use can appear on the surfaces. These are more or less noticeable depending on light incidence. They do not constitute a lessening of utility. As is the case with all surfaces over the long term, due to the wear involved in the daily routine of the medical practice, they cannot be avoided. Spots or stains caused by liquids that may corrode the surface (for example, propolis, tea tree oil, cleaners that contain chlorine or similar substances) are categorically excluded from the warranty.

### Risk of disease transmission

Insufficient disinfection can lead to transmission of disease. Always comply with the requirements of the responsible national body for hygiene and disinfection. Disinfect the product on a regular basis (see notes on disinfection and care).

### Warranty

The product comes with a 2-year warranty. Normal wear of surfaces, castors, handles, etc., and other moving parts is excluded from the warranty. Traces of use and signs of wear and tear due to age and service life, which are based on normal use corresponding to the product performance and the nature of the product, are in accordance with the contract and are not defects in the legal sense.

The warranty is valid from the date of purchase when the product was first bought. The date of purchase must be confirmed by the sales receipt. Within the scope of the warranty, HELLMUT RUCK GmbH will, at no charge, either repair or replace the defective products – at RUCK's discretion.

The prerequisite is carriage-paid shipment of the defective product together with the purchase receipt. The local agent is responsible for claims made outside of Germany. In case of doubt, please check with RUCK before returning the device.

The warranty becomes null and void if we find that the problem was caused by improper or incorrect installation/assembly or usage, by non-observance of the instructions of use, by external intervention or as a result of an unauthorised repair or modification.

The warranty is limited to repair or replacement of the product. Further liability (particularly loss of revenue or earnings) is excluded. However, if it transpires that it is a problem that is not covered by the terms of the warranty or that the period of the guarantee has elapsed, any costs for inspection and repair must be borne by the customer.

We reserve the right to make changes to and deviations from the technical design.

### Scope of delivery

- > RUCK® Foot Bath Trolley ROOM POOL
- > Foot Bath for RUCK® Foot Bath Trolley ROOM POOL (Item no. 1014501)

### If you have any questions, please contact our SERVICE department

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© 2023 HELLMUT RUCK GmbH. All rights reserved. This document may not be copied, edited, reproduced or translated without the prior written consent of HELLMUT RUCK GmbH.