
Connector Set

for RUCK® ROOM move/light

Instructions for Use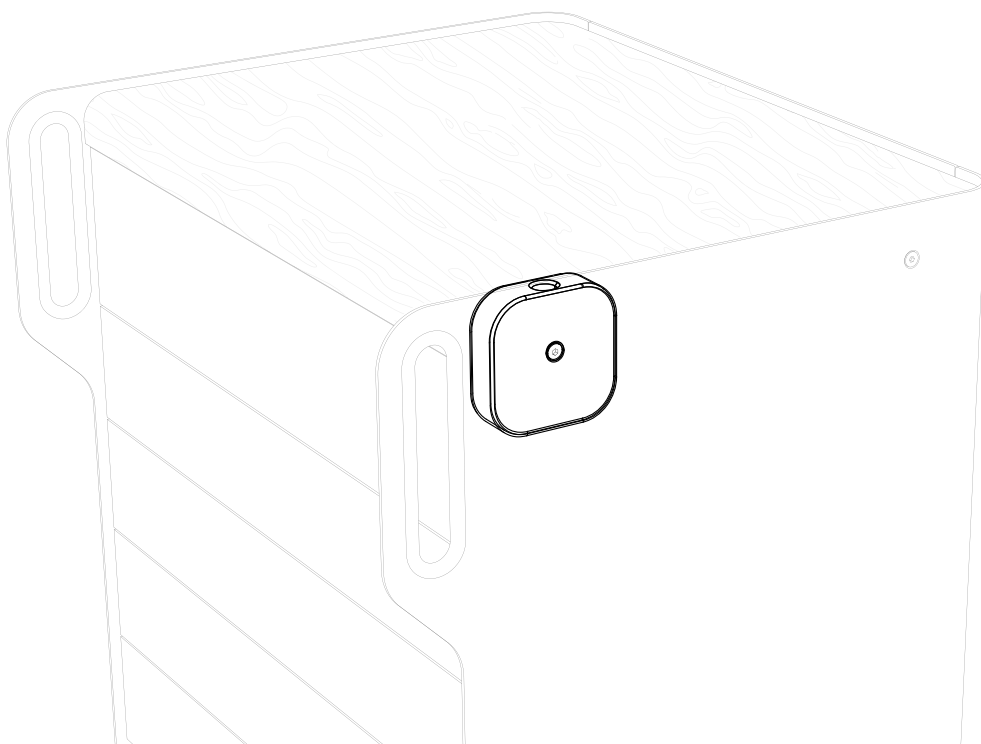

Item no.
1092801

RUCK® Connector Set for RUCK® ROOM move/light

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| |
|--|
| Downloads |
| > Instructions for use |

Copyright

© 2023 HELLMUT RUCK GmbH. Alle Rechte vorbehalten. All rights reserved. This document must not be copied, edited, reproduced or translated without the prior, written permission of HELLMUT RUCK GmbH.

Contents

Abbreviations and symbols 4

Declaration relating to these instructions for use 4

Scope of delivery..... 5

Video assembly instructions..... 6

Assembly instructions 7

Matching accessories 20

Spare parts..... 21

Disposal 21

Warranty 22

Abbreviations and symbols

This symbol identifies a potential point of danger for people or for the device itself. It must be taken note of.


This symbol identifies useful advice. Additional information about the product and how to use it is provided here.

Declaration relating to these instructions for use

Hellmut Ruck GmbH reserves all rights to modify and supplement these instructions for use. Hellmut Ruck GmbH has taken great care to keep these instructions free of inaccuracies and omissions. Please contact us if you find mistakes or inaccuracies. However, Hellmut Ruck GmbH assumes no liability for mistakes in these instructions for use or consequential damage resulting from the provision of these instructions for use or their application when using the product.

These instructions for use must not be disseminated or supplemented without the express approval of Hellmut Ruck GmbH.

Scope of delivery

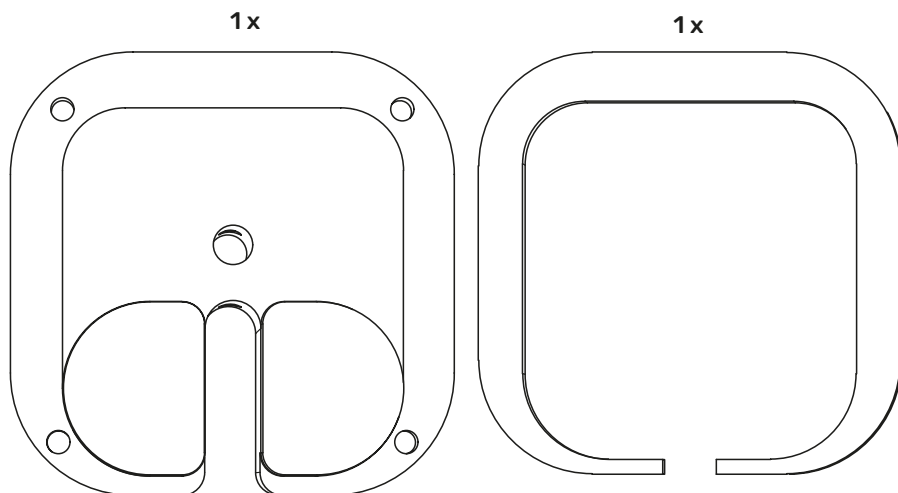
1x 1201501, spacer bush for LUXO


1x 1201502, spacer bush for Daylight iQ

1x 9692901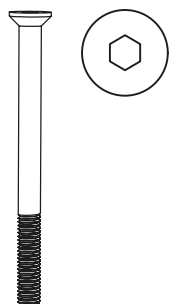


1x Allen key, size 4 mm

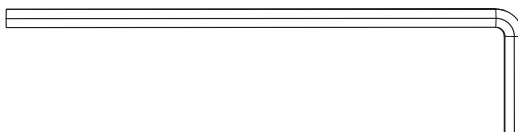Video assembly instructions


For detailed instructions on how to install the Connector Set for RUCK® ROOM light/move, find the video by scanning the QR code or by typing the following link into your browser.
<https://vimeo.com/372833211>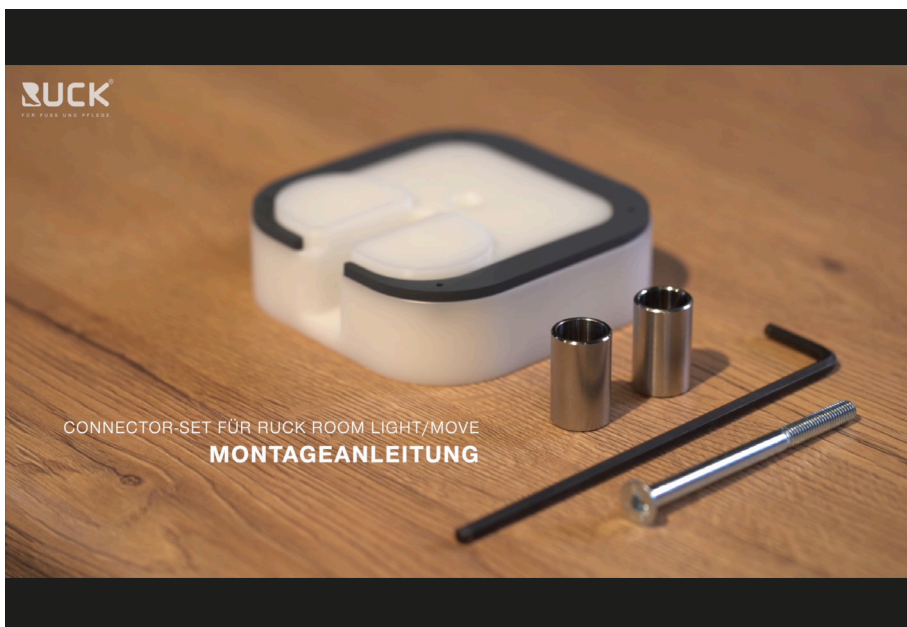


Assembly instructions


①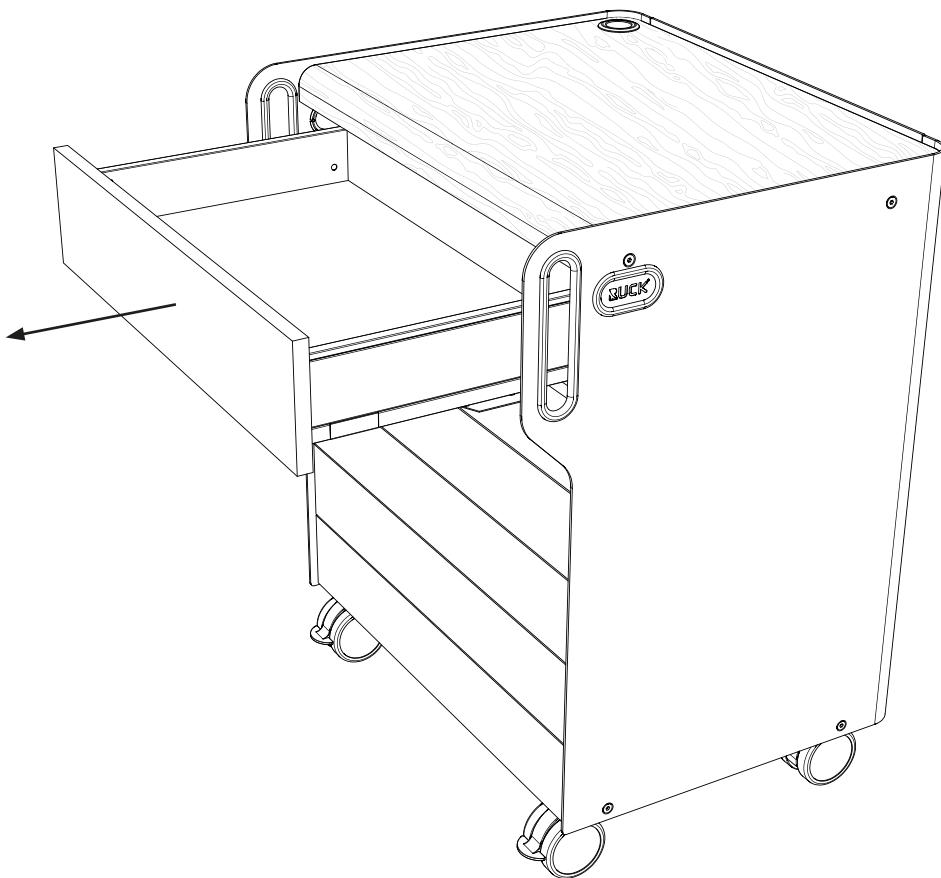


②

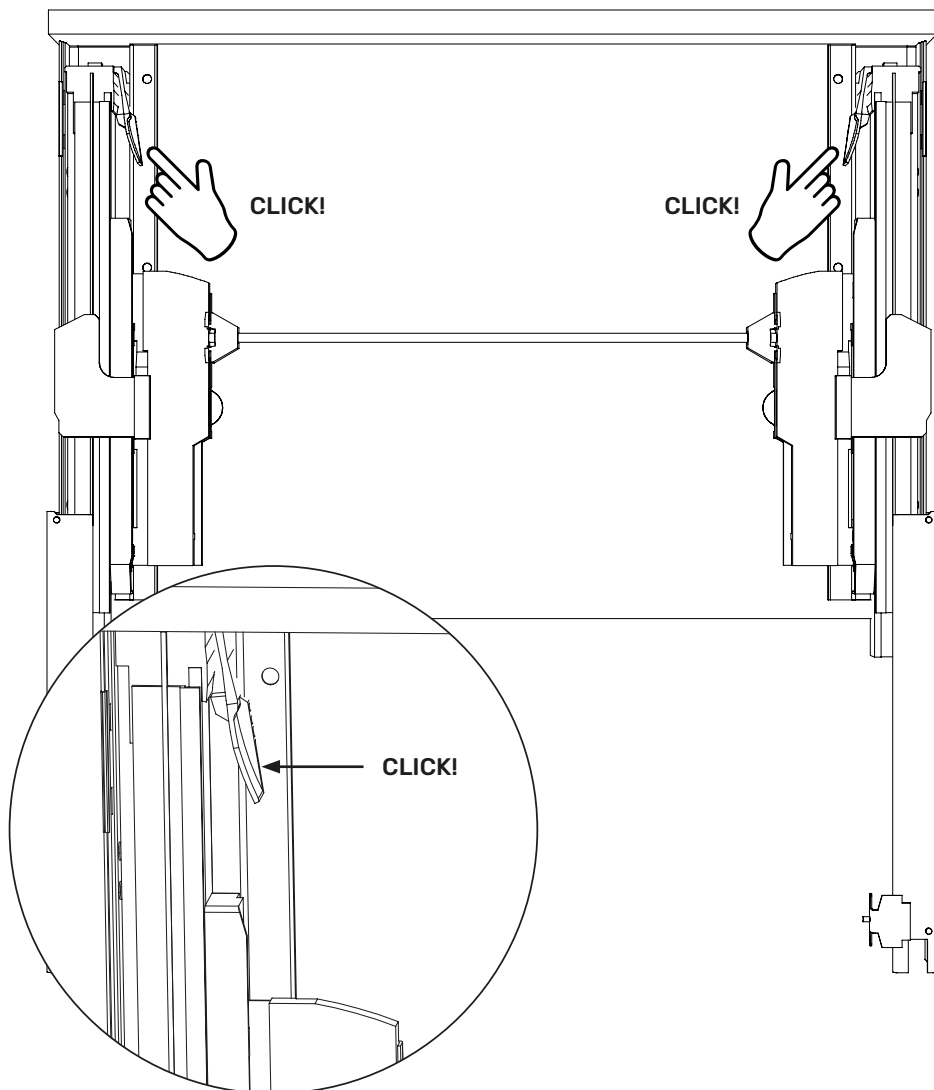
3

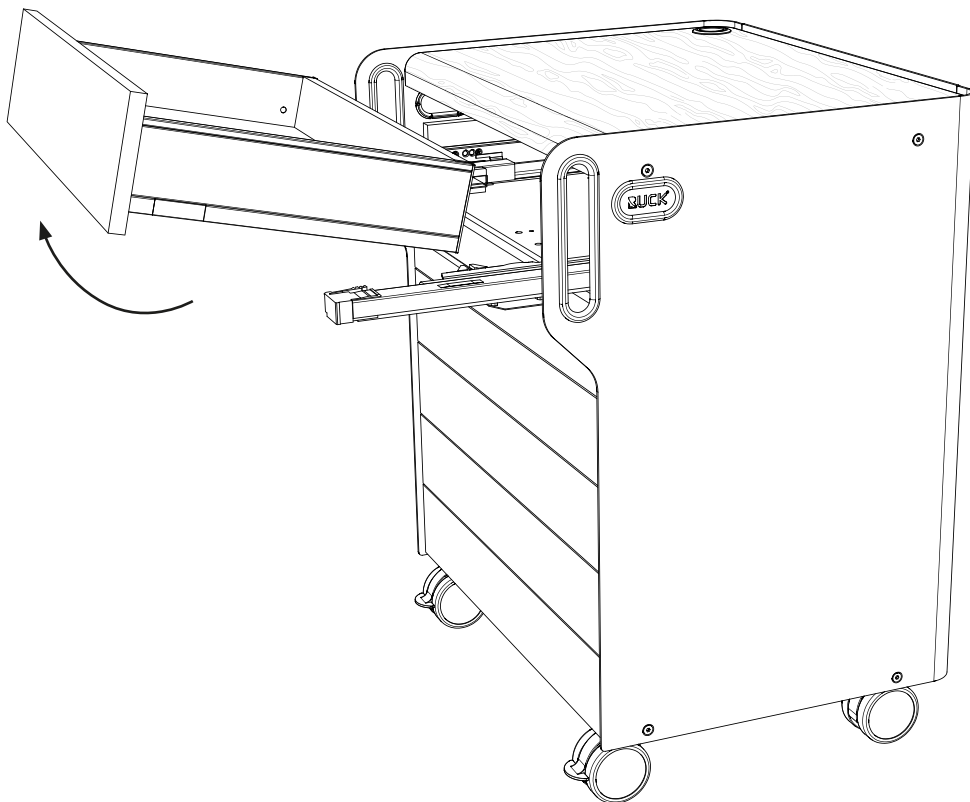
4

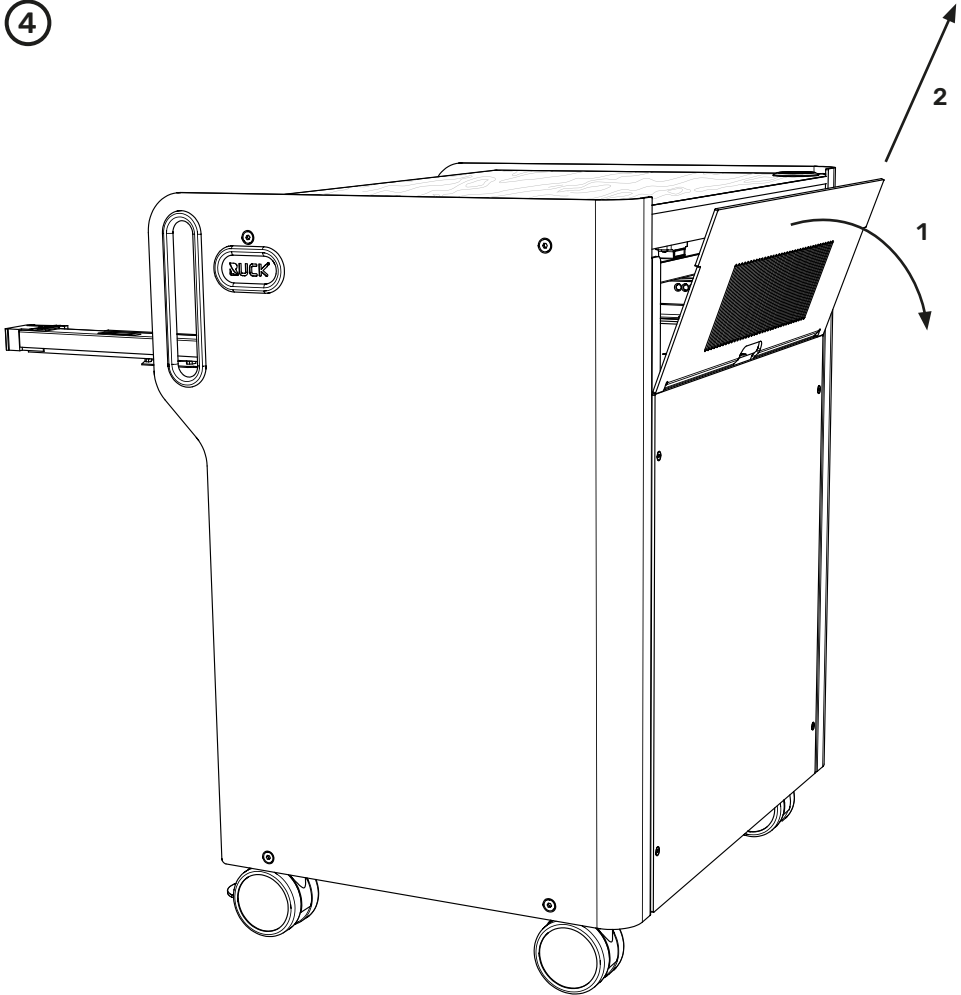
5

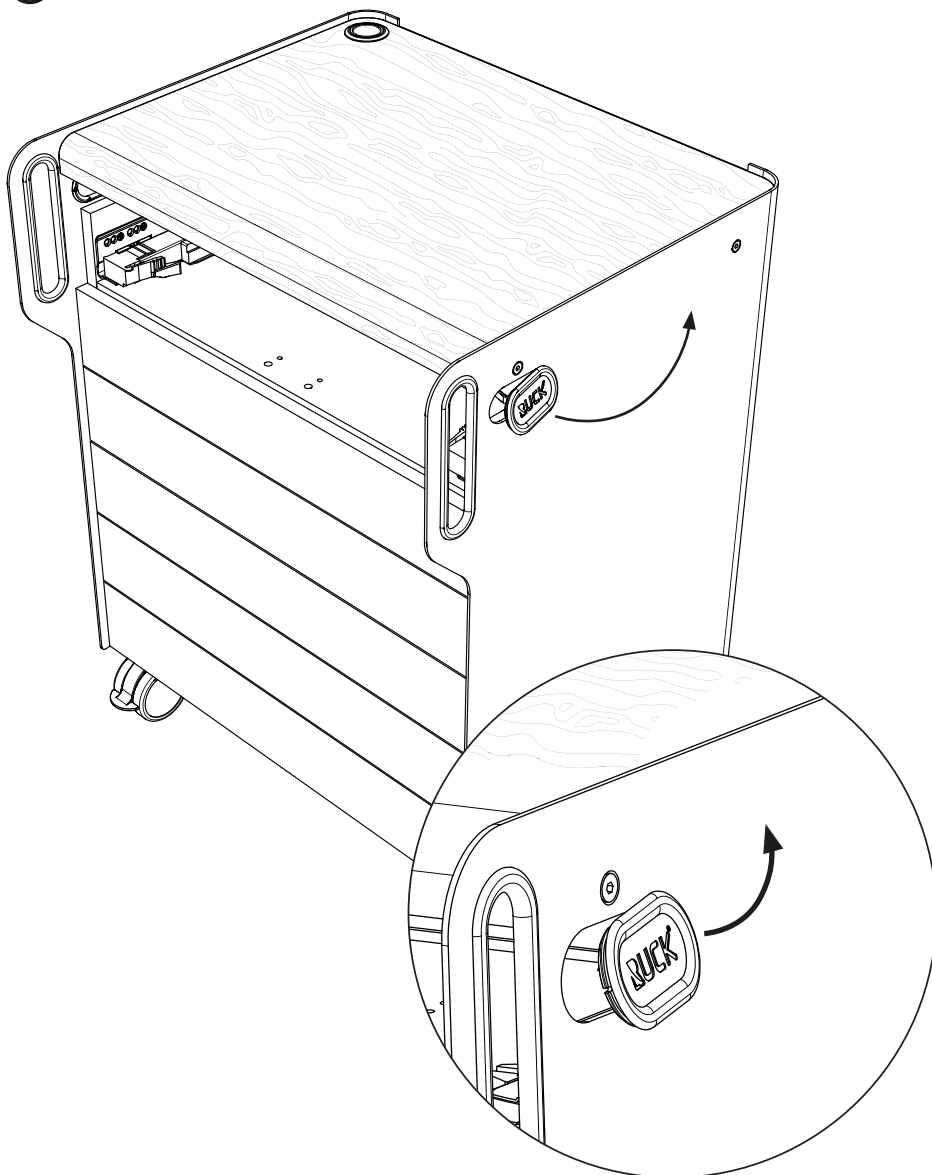
6

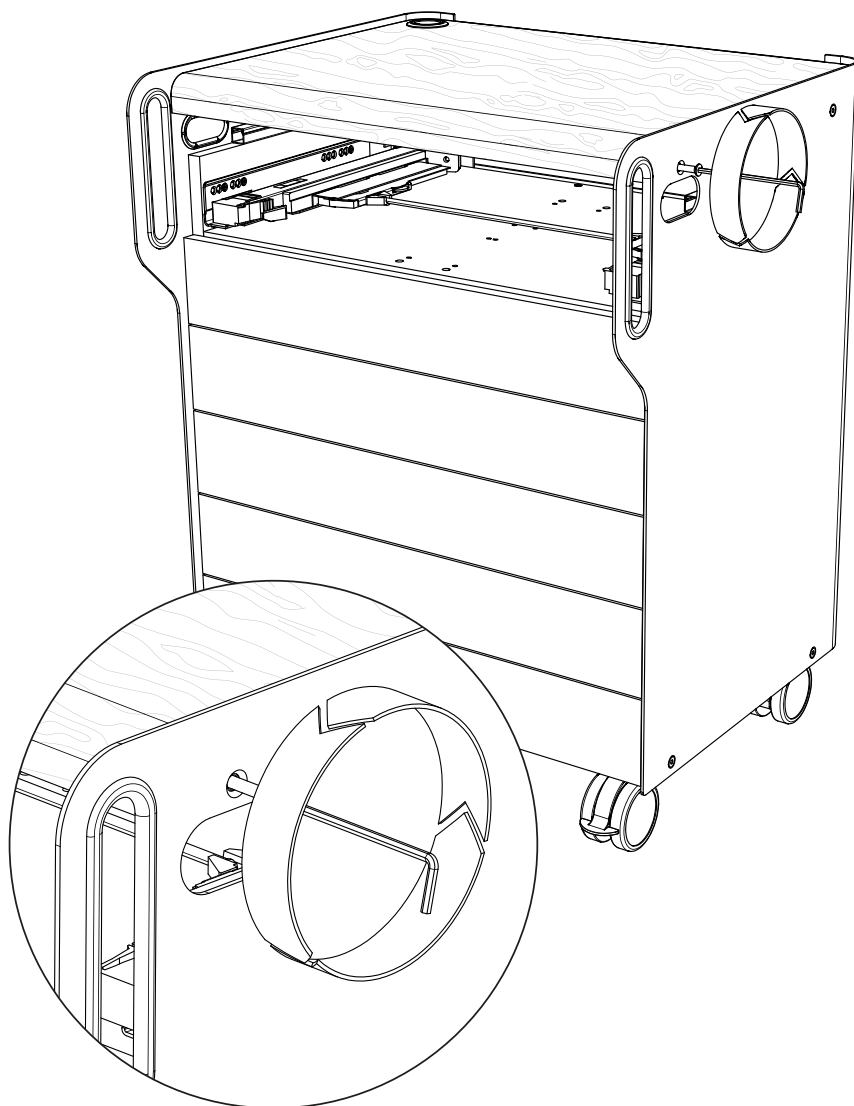
7

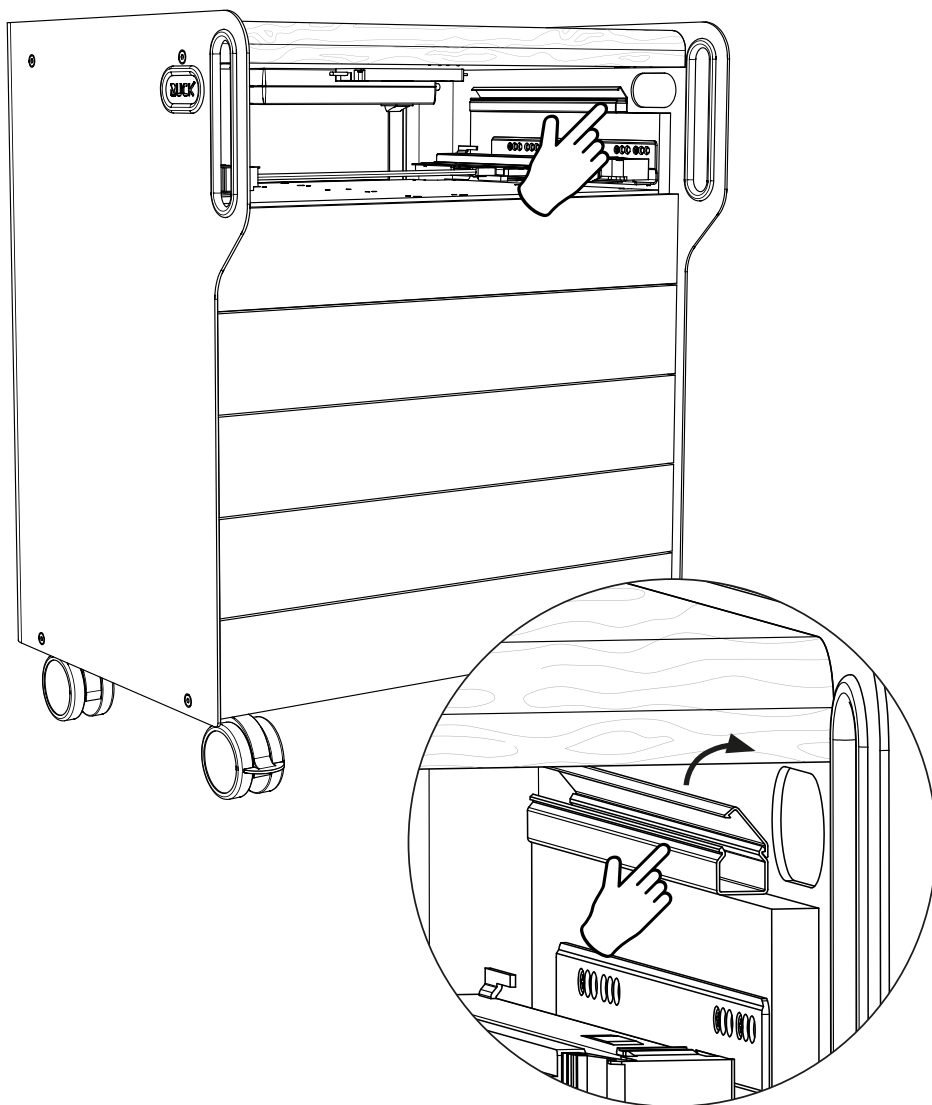
8

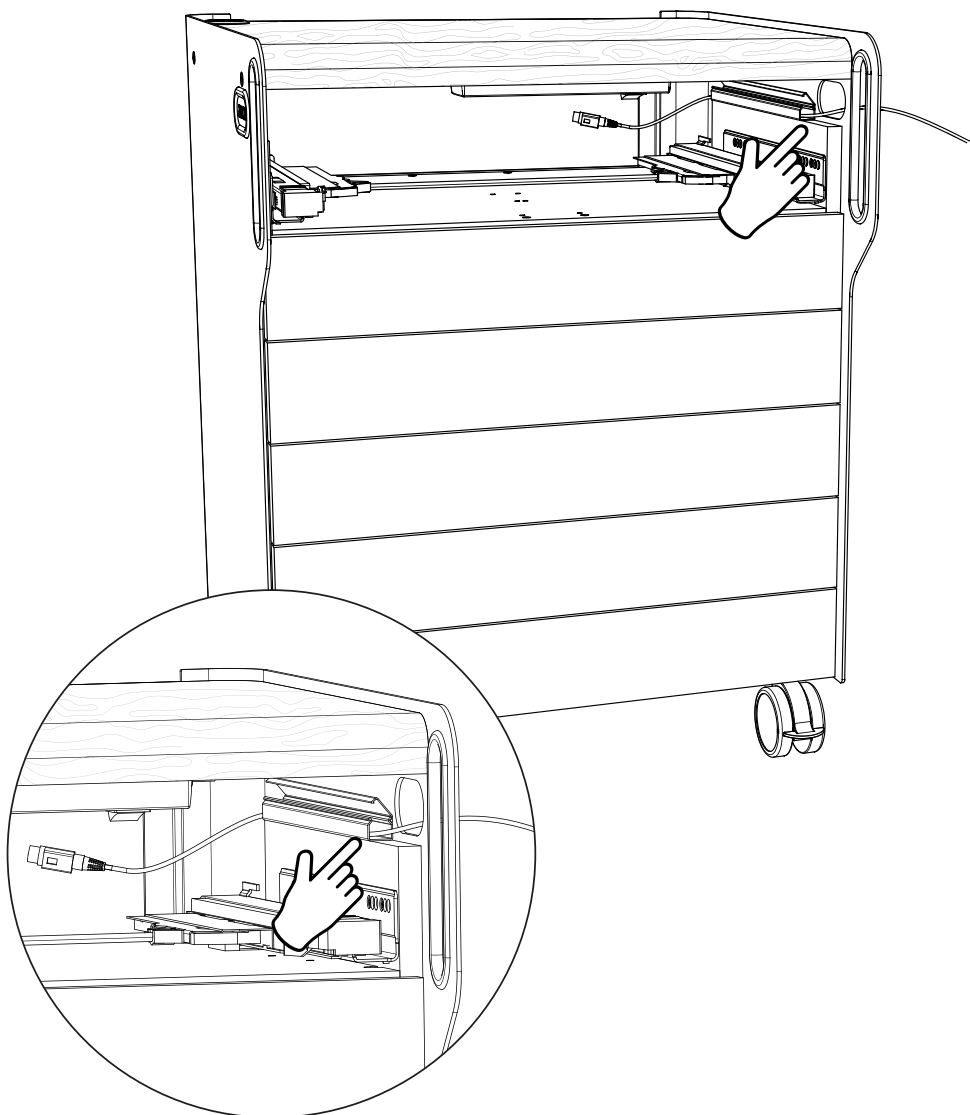
9

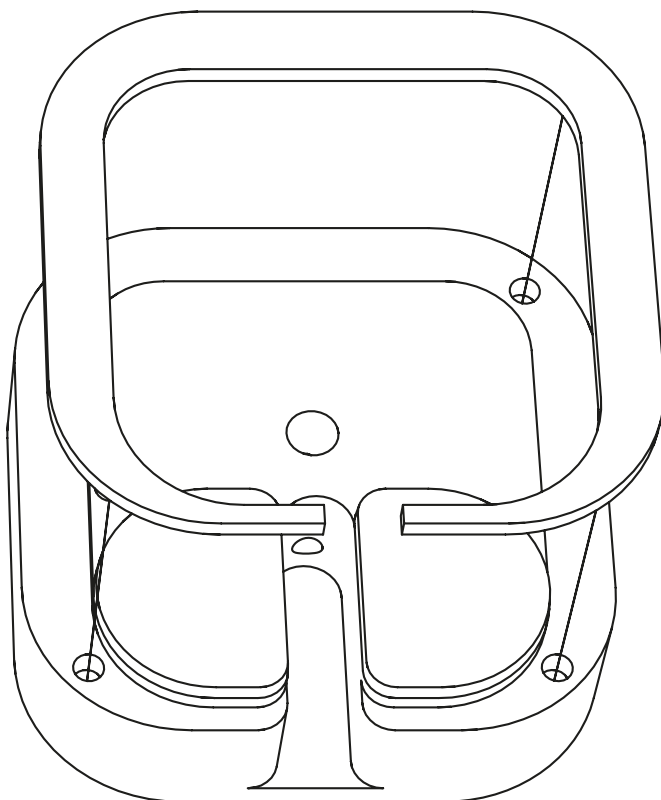10

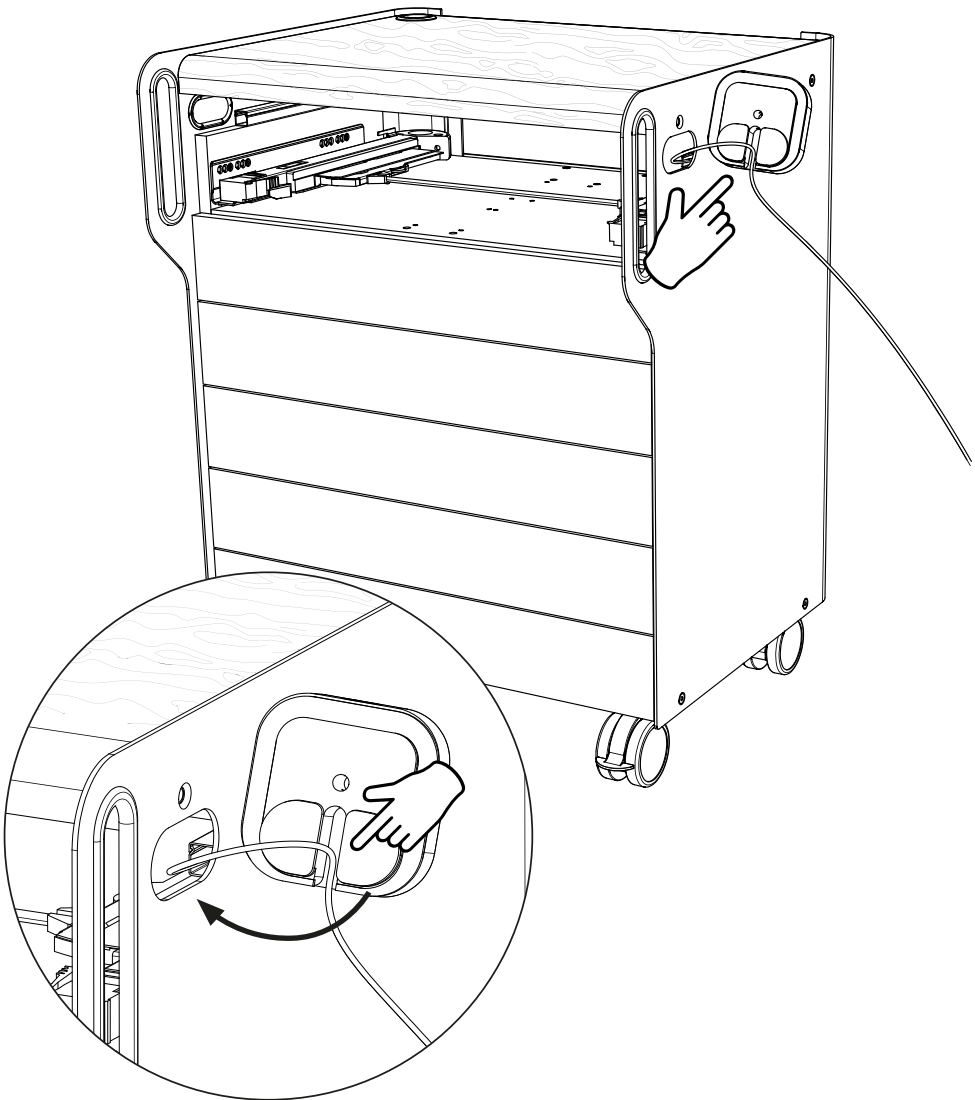
11

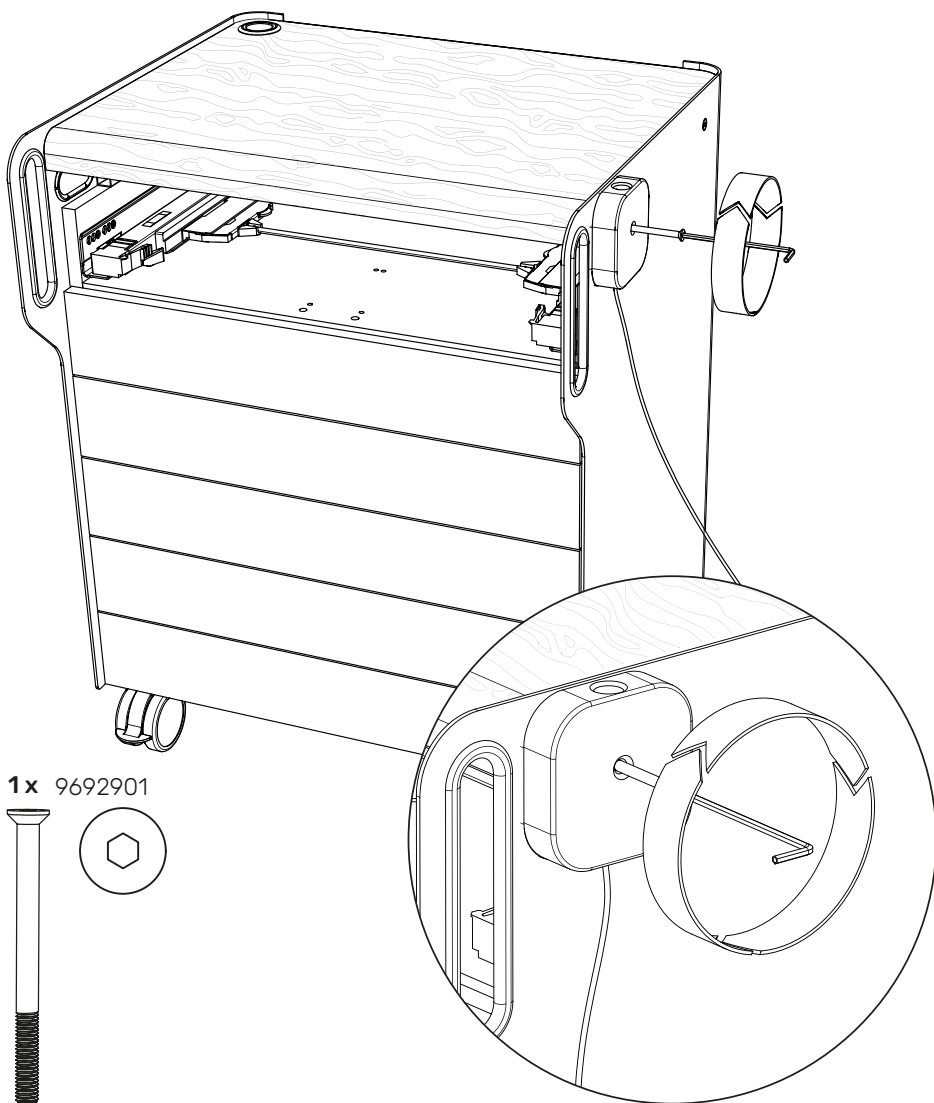12 a


1205401 | PODOLOG CIRCLE XL classic, BLACK EDITION

1205402 | PODOLOG CIRCLE XL classic, WHITE EDITION

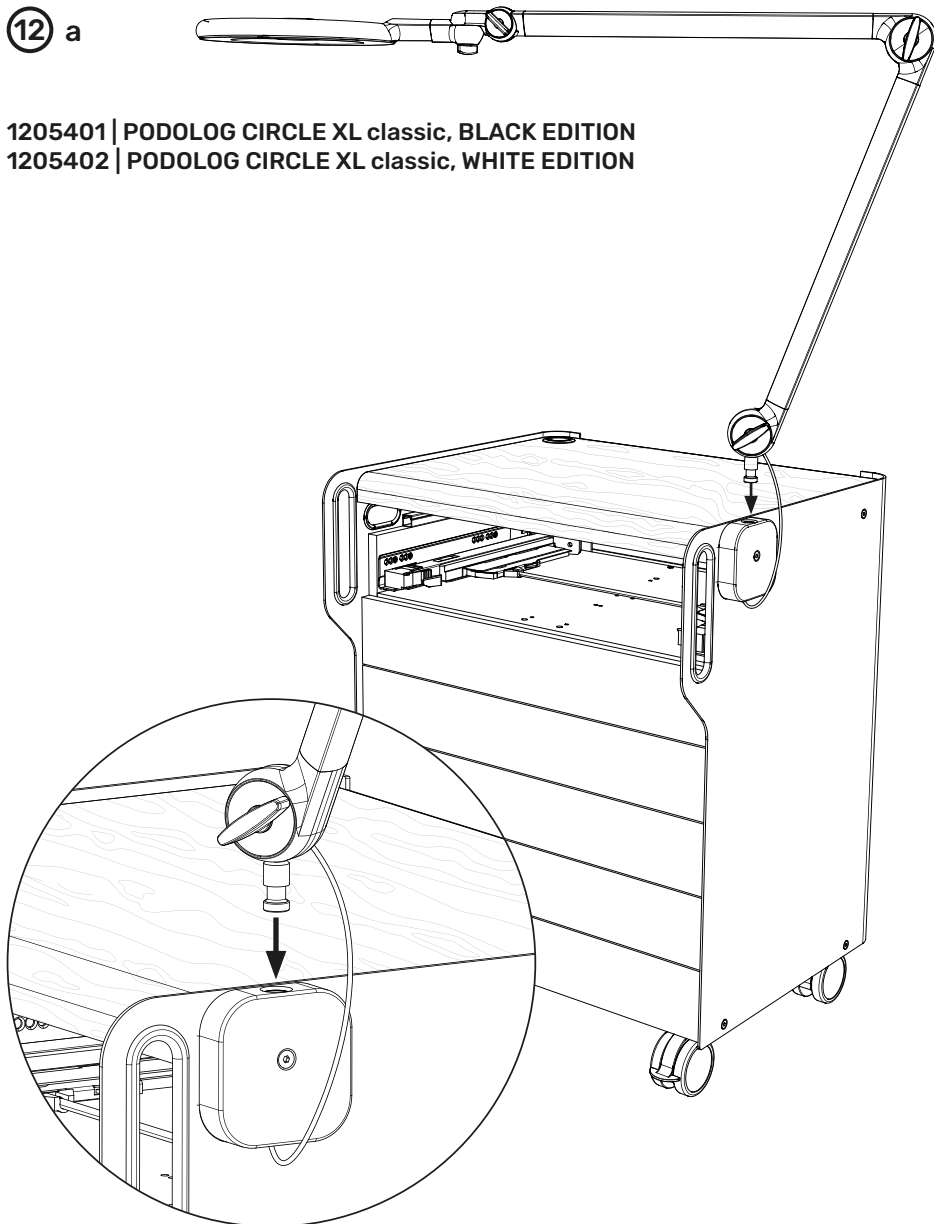
12 b

1200501 | LUX0 magnifier lamp, Circus LED
1207001 | Daylight iQ LED magnifier lamp, white/grey

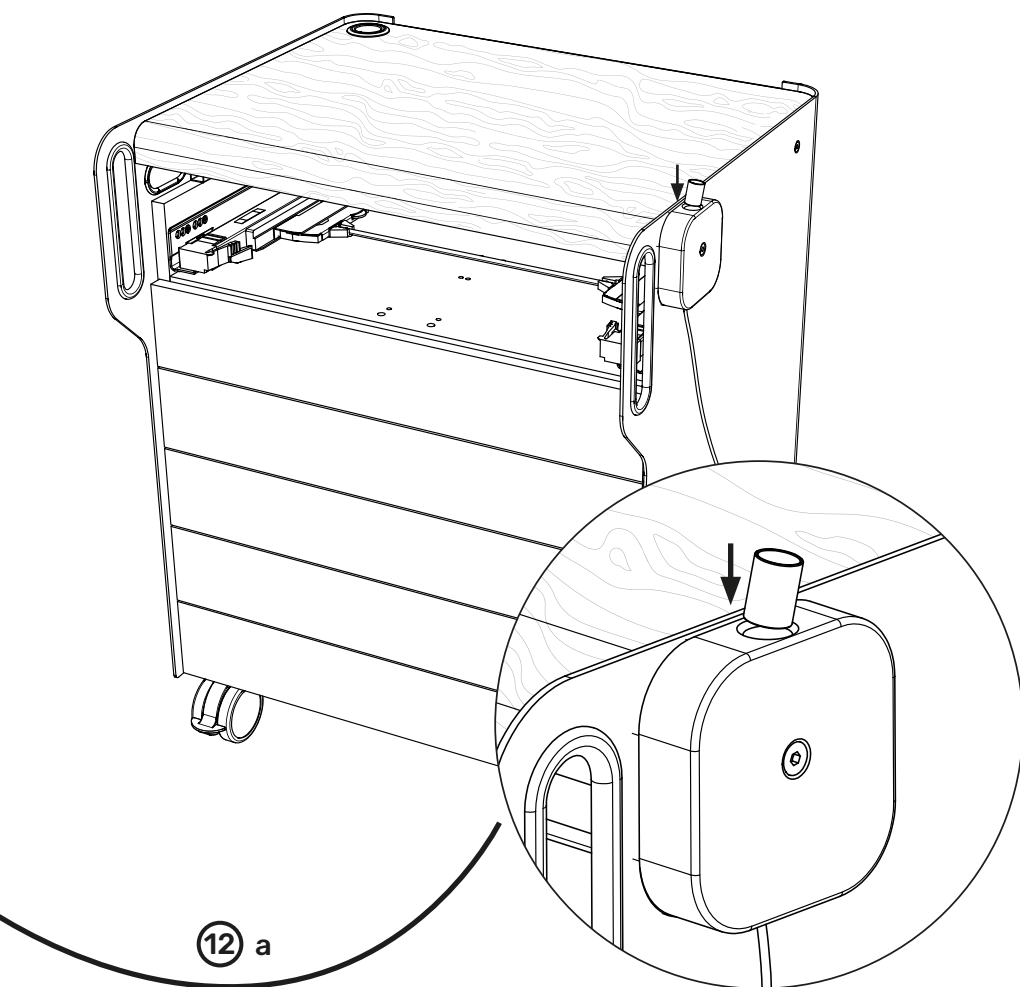
12 a

Matching accessories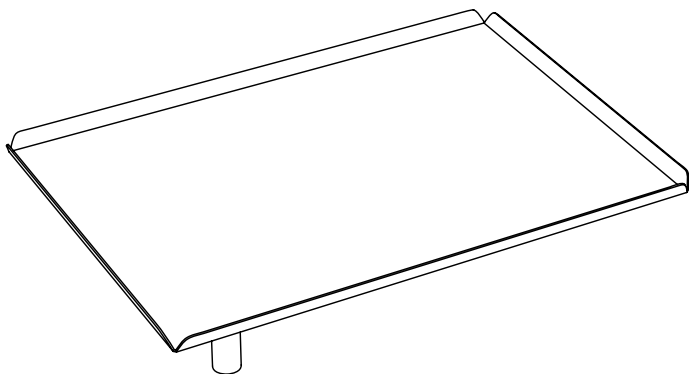

1021501 | Stainless steel instrument tray, swivelling

W/D: 33,5/23,5 cm

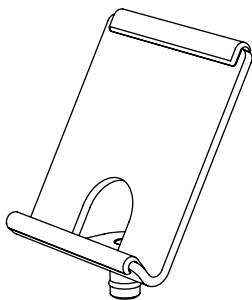1046601 | pododesk tray holder for RUCK® ROOM furniture

Spare parts

Please contact **RUCK SERVICE** for spare parts.

Telephone +49 (0)7082. 944 22 55

Fax +49 (0)7082. 944 22 59

E-mail service@hellmut-ruck.de

Disposal

Environmental hazard!

Incorrect disposal endangers our environment

The RUCK® Connector Set for RUCK® ROOM light/move must be disposed of through available return and collection systems at the end of its service life!

Warranty

The product comes with a 2-year warranty. Normal wear of surfaces, castors, etc., and other moving parts is excluded from the warranty. Traces of use and signs of wear and tear due to age and use, which are based on normal use corresponding to the performance values and the nature of the RUCK® Connector Set for RUCK® ROOM light/move, are in accordance with the contract and are not defects in the legal sense.

The warranty runs from the date of purchase when the product was first bought. The date of purchase must be confirmed by the sales receipt. Within the scope of the warranty, HELLMUT RUCK GmbH will, at no charge, either repair or replace the defective products – at RUCK's discretion. The prerequisite is carriage-paid shipment of the defective product together with the purchase receipt. The local agent is responsible for claims made outside of Germany. In case of doubt, please check with RUCK before returning the device.

The warranty becomes null and void if we find that the problem was caused by improper or incorrect installation/assembly or usage, by non-observance of the instructions for use, by external intervention or as a result of a non-authorized repair or modification.

The warranty is limited to repair or replacement of the product. Further liability (particularly loss of revenue or earnings) is excluded. If however, it turns out that it is a problem that is not covered by the terms of the warranty or that the period of the guarantee has elapsed, any costs for inspection and repair must be borne by the customer.

We reserve the right to make changes to and deviations from the technical design.

8085502 | Instructions for Use RUCK® Connector Set | REV 03 |
Revision status 26.04.2023

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