



**RUCK**<sup>®</sup>  
FÜR FUSS UND PFLEGE

## MOON treatment chair


TRANSPORT LOCK



**WARNING!!!**

PLEASE READ THESE INSTRUCTIONS ON HOW TO REMOVE THE TRANSPORT LOCK **BEFORE COMMISSIONING** AND CAREFULLY CARRY OUT THE STEPS DESCRIBED IN THEM!!!

**IF THESE INSTRUCTIONS ARE NOT HEEDDED, YOUR TREATMENT CHAIR COULD BE DAMAGED, THE GUARANTEE WILL LAPSE AND WE CANNOT ACCEPT LIABILITY.**

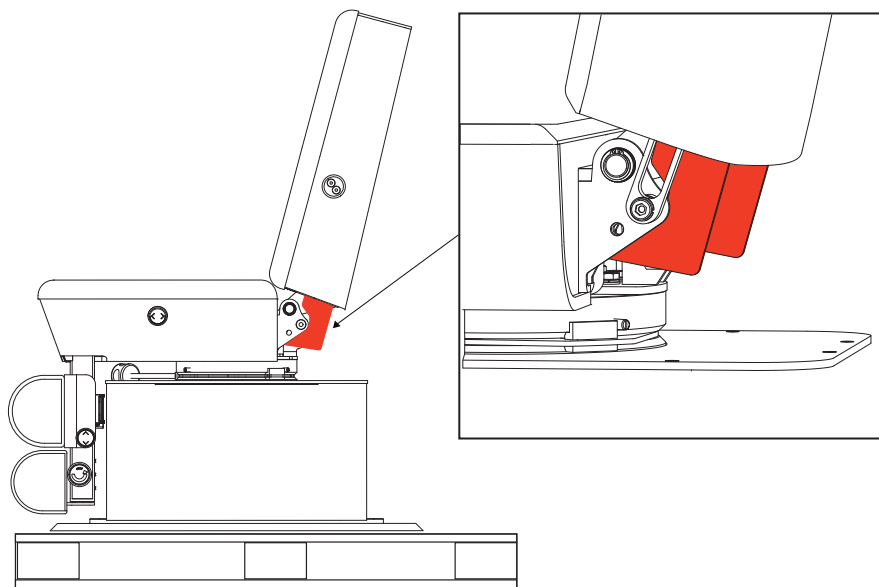

## DECLARATION RELATING TO THESE INSTRUCTIONS FOR USE

Hellmut Ruck GmbH reserves all rights to modify and supplement these instructions for use. Hellmut Ruck GmbH has taken great care to keep these instructions free of inaccuracies and omissions. Please contact us if you find mistakes or inaccuracies. However, Hellmut Ruck GmbH assumes no liability for mistakes in these instructions for use or consequential damage resulting from the provision of these instructions for use or their application when using the product.

These instructions for use must not be disseminated or supplemented without the express approval of Hellmut Ruck GmbH.

## SYMBOLS USED



This symbol identifies a potential point of danger for people or for the device itself. It must be taken note of.



This symbol identifies useful advice. Additional information about the product and how to use it is provided here.



HELLMUT RUCK GmbH | Daimlerstraße 23 | D-75305 Neuenbürg, Germany  
web [www.hellmut-ruck.de](http://www.hellmut-ruck.de) | tel. +49 (0)7082. 944 20 | fax +49 (0)7082. 944 22 22

## Copyright

© 2020 Hellmut Ruck GmbH. All rights reserved. This document must not be copied, edited, reproduced or translated without the prior written permission of Hellmut Ruck GmbH.




## WARNING!

The treatment chair may only be moved by the two armrest mountings on the side of the back section using the transport lock fitted. When transporting, setting up and fitting the levelling pads under the chair, the two red transport locks must be fitted on the treatment chair.


Keep the transport locks for subsequent moves. You can use them to secure the chair again should you wish to change its position at the installation site. We are happy to advise you on transporting your treatment chair as part of a move.

Failure to follow this advice may result in damage to the treatment chair.


**IF THESE INSTRUCTIONS ARE NOT HEHEDED, THE GUARANTEE WILL LAPSE AND WE CANNOT ACCEPT LIABILITY!**

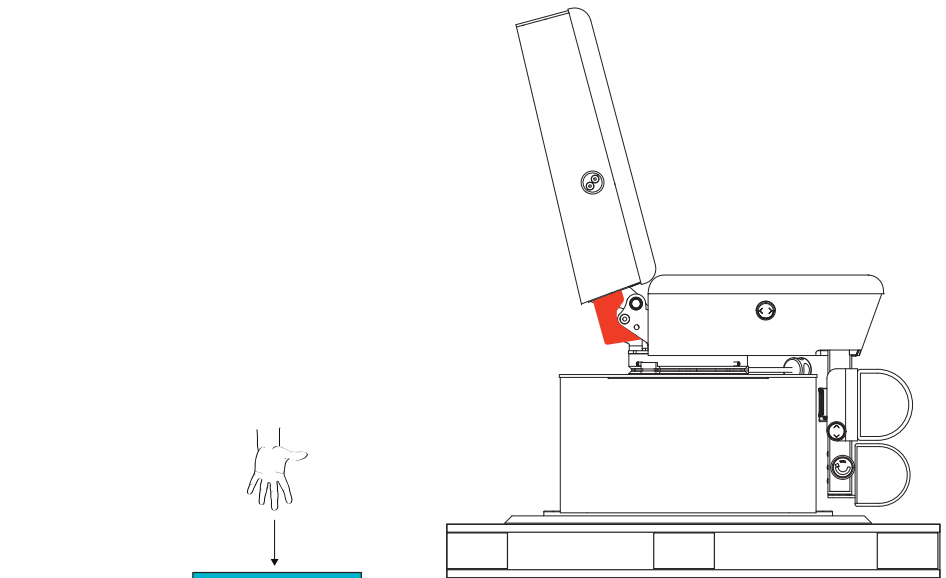


1. Before unloading the treatment chair, place a protective mat (e.g. foot mat) in front of the pallet to protect the floor.

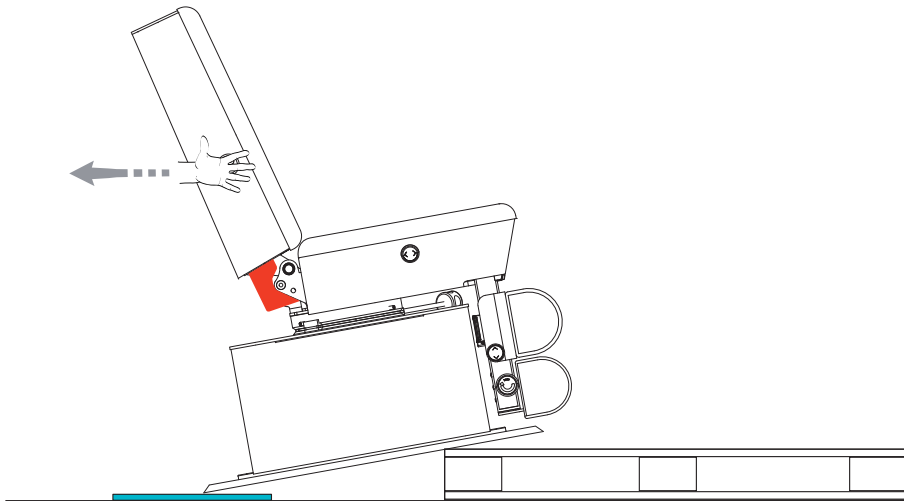


2. Grip the two armrest mountings on the right/left of the back section as shown. Pull the treatment chair off the pallet in the direction indicated by the arrow until the floor plate is sitting on the protective mat.

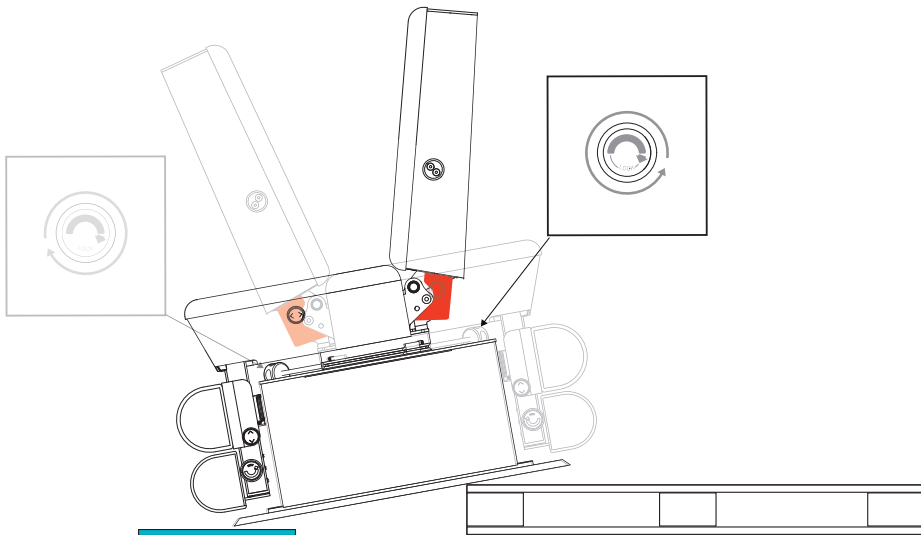


3. Turn the treatment chair by 180° so that the foot supports do not come into contact with the pallet when lowering. To do this, turn the hand wheel for rotation.

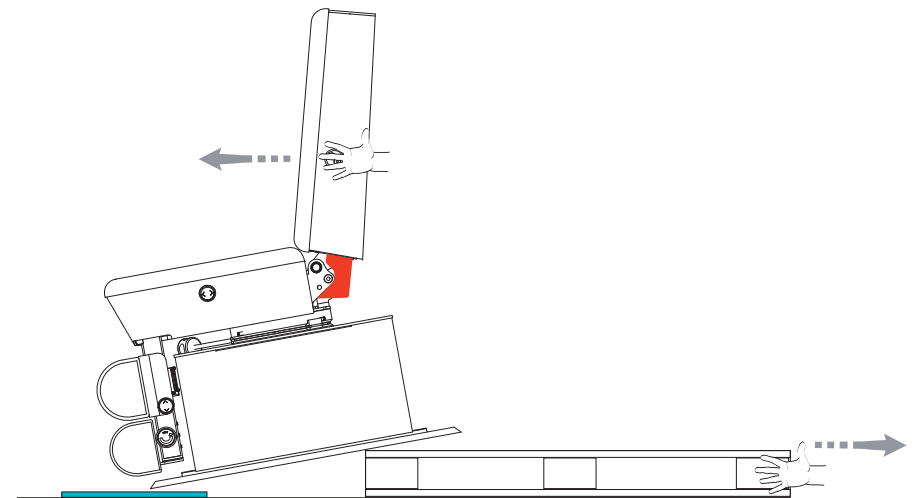


4. Grip the two armrest mountings on the right/left of the back section as shown. Lift the treatment chair and have a second person pull out the pallet from underneath.

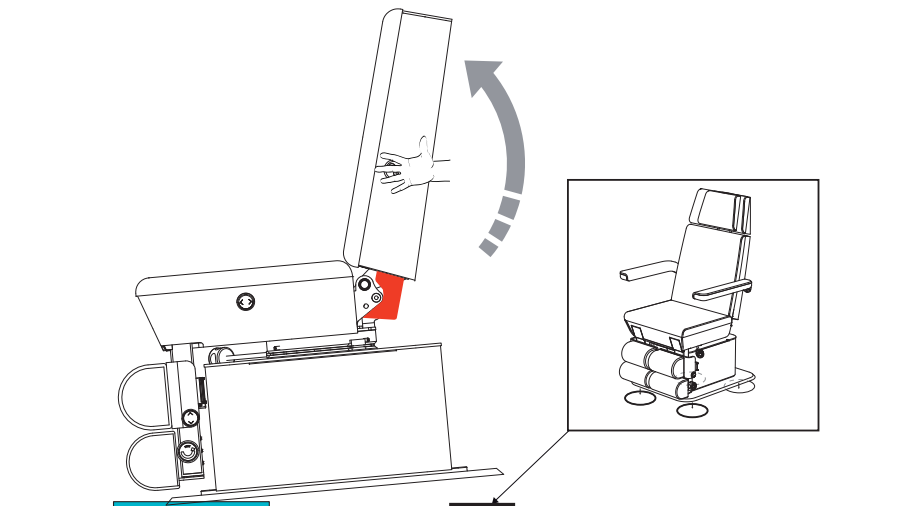


5. Grip the two armrest mountings on the right/left of the back section as shown. Lift the treatment chair and have a second person place two levelling pads under the floor plate.

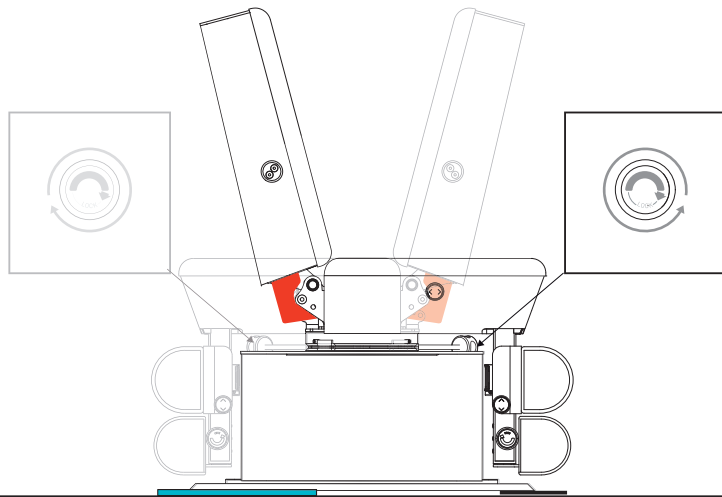


6. Turn the treatment chair by 180°.

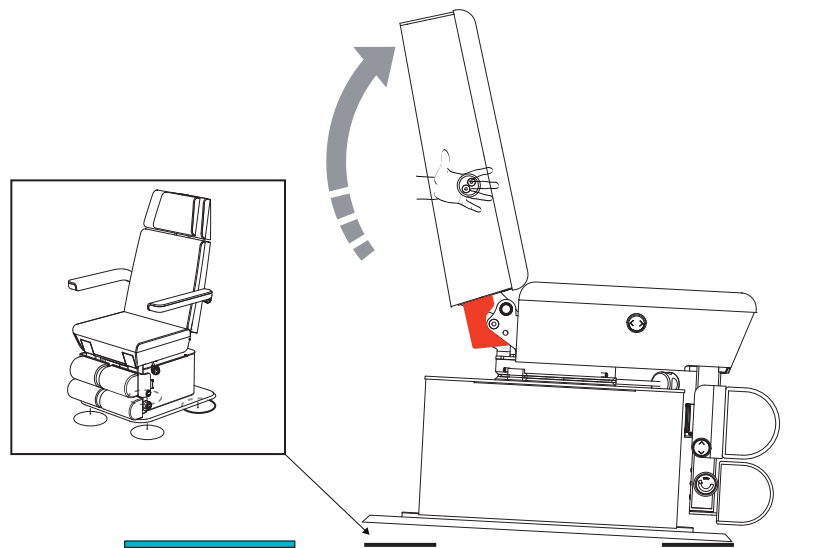

7. Grip the two armrest mountings on the right/left of the back section as shown. Lift the treatment chair and have a second person place two levelling pads under the floor plate.

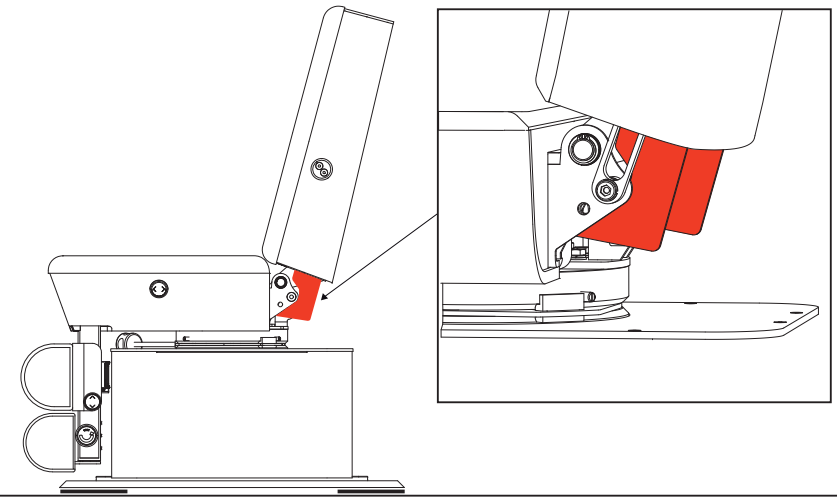


8. The two red transport locks must be removed before commissioning the treatment chair. Failure to do so may mean that the back section cannot be tilted, the drive and/or seat and back section may thereby be damaged.

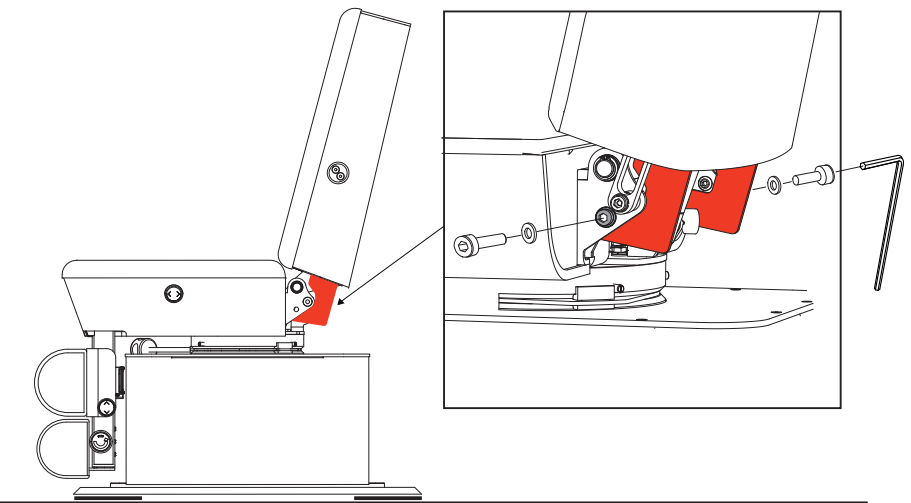

9. Before starting to remove the transport lock, protect the wood cover of the lift cladding and the floor plate of the treatment chair. Use a socket head wrench to loosen the hexagon socket screw on the right/left and remove with the corresponding washer.

# MOON treatment chair TRANSPORT LOCK

8 |

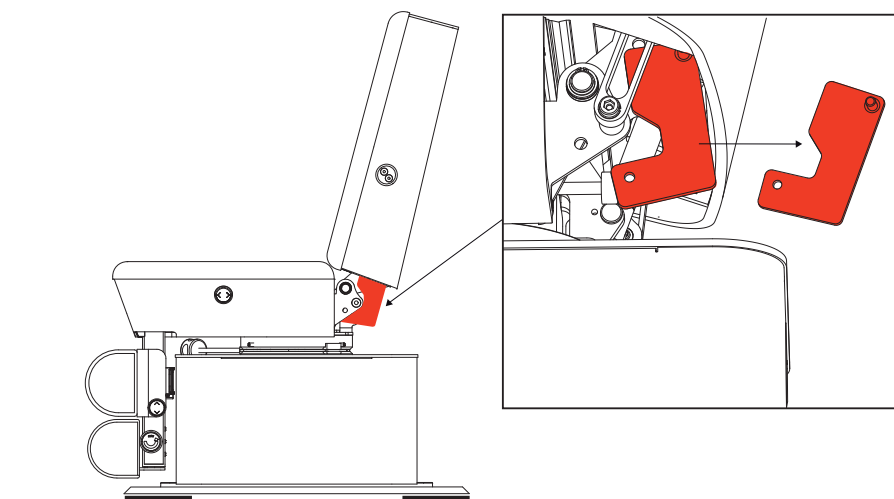

10. The transport lock can now be swung down and taken off with ease.



Keep the transport locks for subsequent moves. You can use them to secure the chair again should you wish to change its position at the installation site. In order to do so, follow points 8-10 in the reverse order.

**HELLMUT RUCK**<sup>®</sup>  
FÜR FUSS UND PFLEGE

HELLMUT RUCK GmbH  
Daimlerstrasse 23  
D -75305 Neuenbürg, Germany  
Tel. +49 (0)7082 944 20  
Fax +49 (0)7082 944 22 22

E-mail [contact@hellmut-ruck.de](mailto:contact@hellmut-ruck.de)  
Website [www.hellmut-ruck.de](http://www.hellmut-ruck.de)